

USER MANUAL

INTRODUCTION


The Wire Loop Game is a tablet game application that motivates and helps with the rehabilitation, specifically hand-eye coordination, of users who experience motor impairment in their fingers and hands. The application challenges the user to follow the path at different speeds and through shapes in increasing difficulty levels.

DEVICE NEEDED TO PLAY

The game application can be run on an Apple iPad device that has the iOS 7 version operating system.

HOW TO START THE GAME

WIRE LOOP GAME

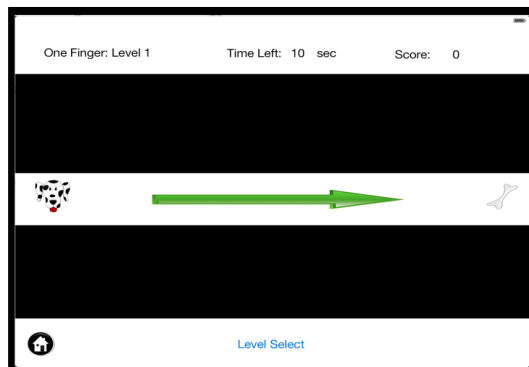

[Tab Here To Start Game!](#)

Figure 1, Starting the Game

Launch the game by tapping the icon. The homepage of the game is the first screen seen. There is a brief instruction on how to play the game and you can tap 'Start Game' button on the screen to start the game.

PLAYING THE GAME

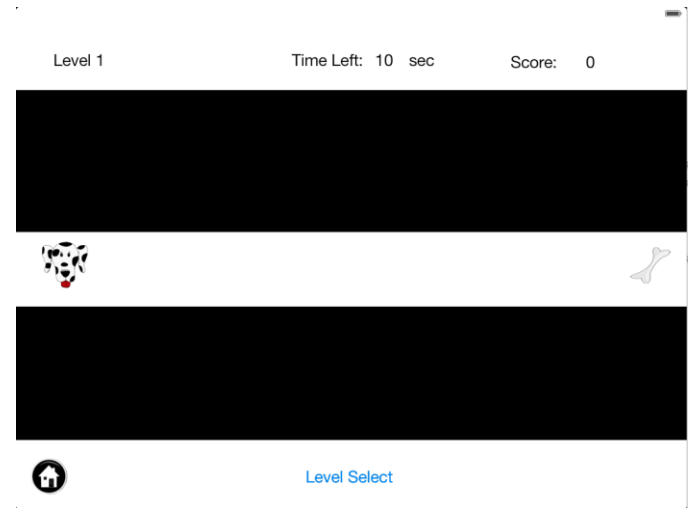


Figure 2, Game Play

Point and drag the dog within the track for reward them at the end with a bone!

Stay precisely within the track to get a better score and the faster you are the better the TIME BONUS becomes, however, if you run out of time you can always try again.

Complete the level within the time to advance to the next one or go back and select a previously played level if you wish to improve your score.

LEVEL CHOICES

Select Level:

1 2 3 4 5



New levels are unlocked after completing the previous one, you can go back and improve your score on any level through the level select menu.

Try your best!